

SURE FOOT

EQUINE STABILITY PADS

FACT SHEET - MEDIUM PADS

Springy

Genuine SURE FOOT Equine MEDIUM Pads are color-coded with a darker Purple DuraFlex top stamped with the official SURE FOOT logo, and are brighter violet colored foam on the other side. They are packaged in pairs along with a Warranty Registration form and QR code for immediate access to the SURE FOOT® Equine website for on-line product registration and education.



This MEDIUM Pad is very springy and is preferred by most people when they stand on SURE FOOT Equine pads. The spring is great for horses that tend to be a bit flat in their movement or are ready for a more challenging experience.

Generally, the MEDIUM Pads would be introduced after the horse has been on HARD or FIRM Pads. The spring can be somewhat off putting for horses that have not been on SURE FOOT pads previously. However, once the horse has been on a more stable pad this is a great choice to challenge your horse's balance a bit more. Mix the MEDIUM Pad in with other SURE

FOOT Equine pads to offer a wide variety of experiences for your horse.

Pads are suitable for use on any surface with horses that are shod, bare foot and/or wearing boots. Pads will show wear over time. Nicks, tears and cuts are considered normal wear and tear and do not limit the effects of the pads in any way.

- **Benefit:** Long-term use to aid in improvement of overall horse health and well-being.
- **Color Coded:** Coated with Purple DuraFlex, a highly durable, anti-microbial coating.
- **Shape:** Rectangular shape allows the different density pads to be paired or stacked for ease of use.
- **Size:** 10"x12"x2" (25cmx30cmx5cm) so that there is sufficient area for the horse's hoof.
- **Weight:** Each pad weighs 1lb. 4 oz. (567 grams).
- **Weight load limit:** Can be used with horses of any size, weight or breed.
- **Safety:** Pads are constructed of material that will not cause injury to the horse. However, failure to follow instructions may cause harm to both horse and human. Always follow the safety guidelines on the back of the insert sheet, website and/or DVD and instructional videos.
- **Instructions for use:** Available on www.surefootequine.com and on YouTube: SURE FOOT Equine Channel. DVD available upon request.
- **Breed or discipline:** May be used with any breed that has a hoof; horse, donkey or mule.
- **Serviceability:** While the pads are not impervious to damage (tears and cuts) they will continue to service the horse even when they look ugly due to use.
- **Portable:** The lightweight pads are easily used in any location. Take them to the arena, competition, field, veterinary clinic or anywhere your horse needs comfort and stability.
- **General cleaning instructions:** Easily cleaned with a soft brush, soap and warm water. Avoid scouring powder or other abrasive cleaners as they may damage the coated side.
- **Health Safety:** Disinfect with soap and water.
- **Hoof Preparation:** Bare foot horses tend to cut the pads so be sure to file off any sharp hoof edges to minimize damage. Shod horses tend to tear the edges of the pad if they slip off.

- **Uses:** Pads are beneficial to the horse pre-exercise, post-exercise, during a training session to improve and enhance performance, recovery from injury while stall bound or in limited movement, reset proprioceptors to establish normal patterns of movement, calming for any breed horse in any discipline, improve overall well-being of horse and owner, improve relationship between horse and owner as the owner is recognized by the horse as caregiver.
- **Durability:** Genuine SURE FOOT pads are durable and have been designed and tested to hold up under the demands of use with horses in any environment. The initial production run in January 2016 are still in use today.
- **Storage** – Be mindful of where you leave your pads. In very hot temperatures Hard pads will soften considerably. Do not leave any SURE FOOT pads in an enclosed car, horse trailer or other confined spaces in hot and/or humid temperatures. Doing so may void the warranty. A rule of thumb is not to leave your pads exposed to direct sunlight for prolonged a period of time. If you wouldn't leave your dog in that location, then don't leave your pads there either!
- **Warranty:** Warranted against delamination of the DuraFlex coating and/or pad layers. Excessive use with multiple horses on a daily basis will cause significantly more damage than the individual horse owner. However, the pads will continue to perform.
- **Manufacturing:** Made of as much environmentally friendly material as possible, designed and tested by Wendy Murdoch, carry the real SURE FOOT Logo and are assembled in the USA under the guidance and direction of The Murdoch Method, LLC.
- **Prop 65 Statement:** For information regarding the State of California Proposition 65 please see the description of the Pads at murdochmethod.com.
- **Warning:** *Please be sure to familiarize yourself with the SURE FOOT® Equine Stability Program before using SURE FOOT pads with your horse. Failure to follow instructions could result in death or serious injury. Check with your veterinarian before using SURE FOOT for any health-related conditions. Keep your hands away from the horse's hoof. Use your foot to position the pads. See instructions for use on the SURE FOOT DVD, SURE FOOT Equine website surefootequine.com and Sure Foot Equine [YouTube](#) Channel.*

